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SUMMER CUSHIONS with Epithelium 26®

Epithelium 26® is a patented silicon gel with mechanical characteristics similar to those of natural plantar footpads.

INDICATIONS :

Open shoes.

Thin or hard plantar tissue due to ageing, calluses, rheumatism, algodystrophy, extended standing, sport hard on the feet (rhythmic gymnastics, etc.).

TIPS FOR USE :

- Place them directly on the skin for optimal comfort (unless you have an unprotected wound).
- First, determine where the right and left foot are. To do so, place the pair of cushions in front of you: the fabric bearing the Epithelium 26® brand should be facing upward, with the writing right-side-up. When you put them side-by-side, if the 2 cushions are properly positioned (the right on the right and the left on the left) you should see a space at the base forming a triangle (see here). Otherwise, reverse them.
- Remove the protective film from the double-sided tape found on the back of the cushions.
- Position the cushions in your shoes, in the widest area. The cushions will be slightly slanting, as shown here.

LASTING QUALITY :

You may machine-wash your cushions (40°C). If you do so, let the cushions air-dry and change the adhesive strip.

CONTRA-INDICATIONS :

- This product has been put through all tests as prescribed by the ISO 10993 standard. In the unlikely event of an allergic reaction, discontinue use and notify MILLET Innovation S.A.
 - Do not use on an open wound.
 - In case of doubt, consult your doctor or your chiropodist.
- MILLET Innovation S.A. cannot be held responsible for use that does not conform with the manufacturer's recommendations.

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