# 3M Opticlude™

Pansements orthoptiques (FR)

## Description:

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# 3M Opticlude™

## Removal

1. At the nose side of eye patch, grasp and loosen the outer edge of the eye patch.
2. Support the skin with your fingers. Remove the tape low and slow, peeling the eye patch towards the ear, keeping it close to the skin surface and pulled back over itself. Please be aware that removing the eye patch at an angle will pull the skin surface and increase the risk of redness and irritation.
3. As the patch is removed, continue to support the newly exposed skin.
4. For a patch that is strongly adhered, you may consider using a medical grade adhesive remover or moisturizer (approved for use around the eye) to soften the adhesive along the peel edge.

# 3M Opticlude™

Orthoptic Eye Patch for occlusion therapy (GB)

## Description:

3M™ Opticlude™ Orthoptic Eye Patch consists of a soft non-adherent pad bonded to a skin-friendly breathable and conformable backing coated with a hypoallergenic adhesive. The dressing is breathable and conforms to body contours.

## Intended Use:

3M Opticlude Orthoptic Eye Patch is used in eye occlusion treatment of Amblyopia and Strabismus in children.

## Precautions:

Follow directions provided by your eye care professional regarding use of 3M Opticlude Orthoptic Eye Patch.

Reused product does not fulfill claims and can not be used as intended.

## Instructions for use:

### Application

1. For best adhesion, first clean and dry skin. Remove paper liner from the eye patch.
2. Gently close both eyes keeping facial muscles relaxed. Wrinkles around the eye should be avoided.
3. Apply 3M Opticlude Orthoptic Eye Patch with the narrow end towards the nose. Do not stretch the patch, or pull skin.
4. Secure the pressure sensitive adhesive tape by applying light pressure with fingertips around the outside of the eye patch.

### Removal

1. At the nose side of eye patch, grasp and loosen the outer edge of the eye patch.
2. Support the skin with your fingers. Remove the tape low and slow, peeling the eye patch towards the ear, keeping it close to the skin surface and pulled back over itself. Please be aware that removing the eye patch at an angle will pull the skin surface and increase the risk of redness and irritation.
3. As the patch is removed, continue to support the newly exposed skin.
4. For a patch that is strongly adhered, you may consider using a medical grade adhesive remover or moisturizer (approved for use around the eye) to soften the adhesive along the peel edge.

### Storage and Shelf Life:

For best results store the 3M Opticlude Orthoptic Eye Patch at normal room temperature. Shelf life is provided by the expiration date printed on the box.

Date of the latest revision of the instruction for use 2010-09

Made in Poland

3M Poland Sp. z o.o.  
Al. Katowicka 117 Kajetany,  
05-830 Nadarzyn

3M and Opticlude are trademarks of the 3M Company  
YP-2763-0231-9/1110