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**PLANTAR CUSHIONS with Epithelium 26®****INFORMATION**

Prevent and alleviate pain in the forefoot.

**PROPERTIES**

- Patented by chiropodists, Epithelium 26® is an effective substitute for the sole of the foot. This silicone gel's mechanical properties allow it to replace the missing plantar tissue that is causing you pain: it is remarkably effective in redistributing your body weight over the whole of your foot.
- The presence of Epithelium 26® in the plantar area allows Plantar Cushions to alleviate pain effectively.

**INSTRUCTIONS FOR USE**

- Wear next to the skin.
- The printed part of the cloth should be in contact with the sole of the foot. Do not turn over.
- Slip your foot into the cushion and place the retaining tab between your big toe and your second toe.
- To help the cushion remain in position, wear tights, socks or stockings over it.

**CLEANING**

- Machine-wash at 40° C in the washing net provided or hand-wash (without scrubbing).
- Leave to dry naturally. Do not iron.

**CONTRAINDICATIONS**

- The product has passed all of the tests required by ISO standard 10993. If, despite this, you suffer an allergic reaction, stop using the product immediately and inform Millet Innovation.
- Do not use on an unprotected wound.
- Persons suffering from arthritis, diabetes, polyneuritis, lower limb neuropathy or varicose ulcers: never use the product on a wound, monitor the condition of your foot during use, and in all cases seek advice from your doctor or chiropodist. The strap may cause friction between the two toes. This solution should therefore be avoided by diabetics suffering from neuropathy.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations.

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